

TROD* TOYO

Anti-HCV

Arm stretched out to the side body (down position), hand at waist level

Tips for drawing blood at the fingertip

Easier sampling by washing hands with hot water


sample collection



1 Identify the TOYO Anti-HCV test then disinfect the finger (avoiding the forceps)



2 Prick your clean finger with the lancing device provided

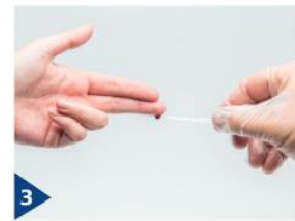

3 Forming a large drop of blood



4 Collect the suspended drop of blood by moving the pipette horizontally towards the drop (do not press on the pipette bulb).



5 Repeat steps 3 and 4 until the blood has reached the black line on the pipette.

Test procedure



1 Squeeze the pipette bulb to empty the blood into the deposit zone



2 Deliver two drops of thinner to the deposit area




3 Read after **15 MIN**
(do not read after 20 minutes)

interpretation of results



disposal zone

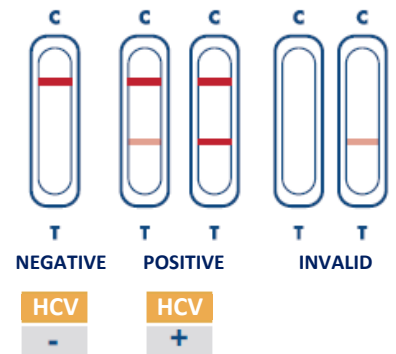

*TROD : Rapid diagnostic test that can be used in accordance with current regulations
THB-Brochure v5 -EN (03/10/2024)